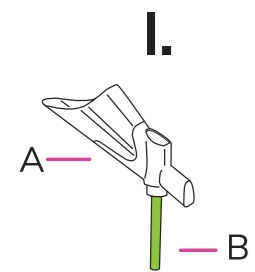
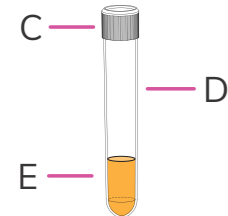
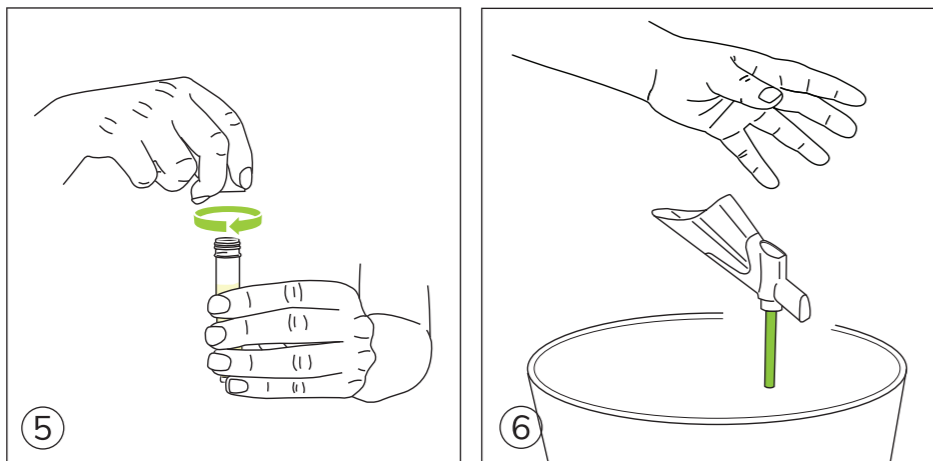
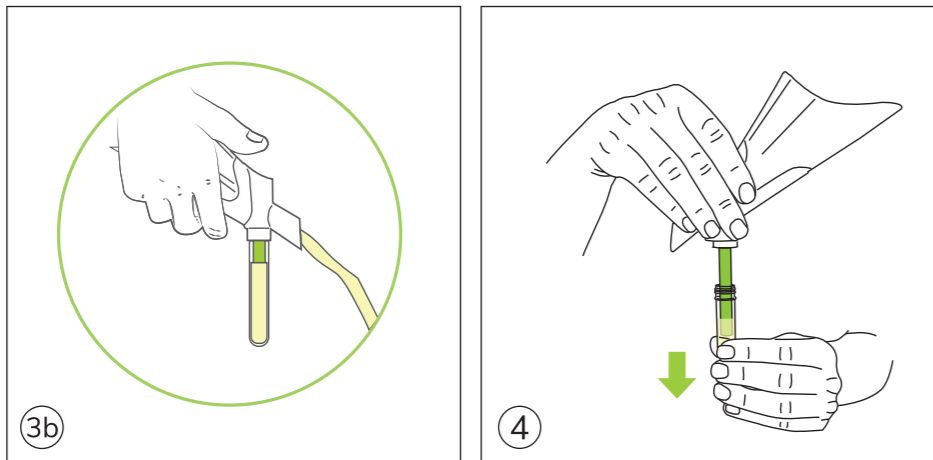
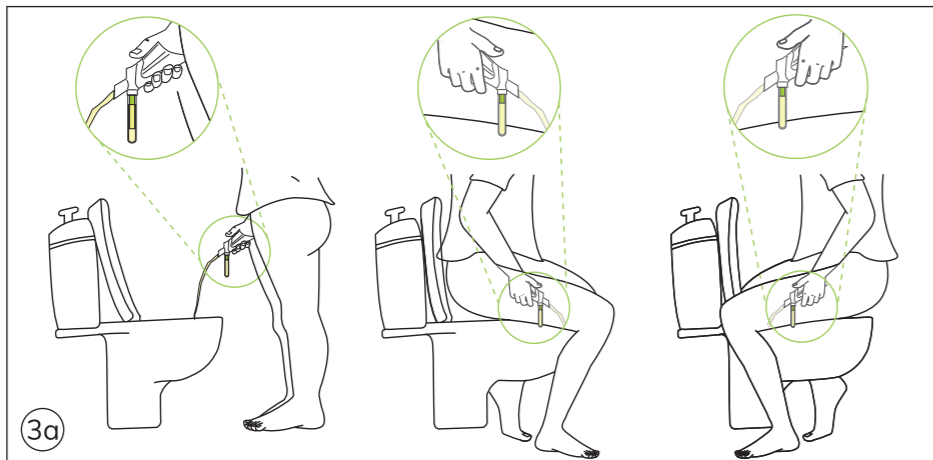
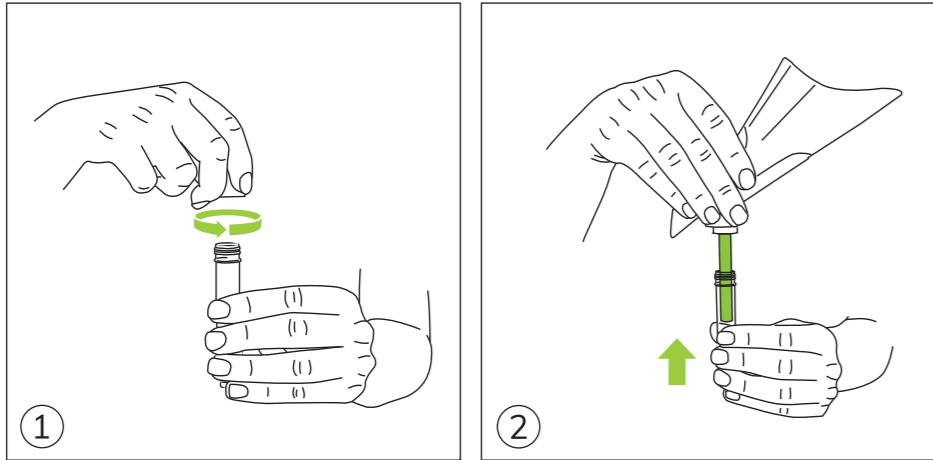
Colli-Pee® UAS™ FV-5010-RUO

REF N00345\_RUO

RUO

COLLI-PEE is a registered trademark in the name of Novosanis NV [www.novosanis.com](http://www.novosanis.com)

II.

Patent ([www.novosanis.com/legal-notice](http://www.novosanis.com/legal-notice))  
© 2022 Novosanis, a subsidiary of OraSure Technologies, Inc., all rights reserved.DNA Genetec,  
3000 – 500 Palladium Drive,  
Ottawa, Canada K2V 1C2Bijkhoevelaan 32c  
B-2110 Wijldegem, Belgium  
[welcme@novosanis.com](mailto:welcme@novosanis.com)  
Subsidiary of OraSure Technologies, Inc.NL  
EERSTE FRACTIE URINE ZELFCOLLECTORVoor gestandaardiseerde, volumetrische verzameling en bewaring van eerste fractie urinestalen.  
Lees vóór gebruik alle instructies.

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\*Meer informatie is te vinden op het SDS, [www.novosanis.com](http://www.novosanis.com)EN  
FIRST-VOID URINE SELF-SAMPLING DEVICEFor standardized, volumetric collection and preservation of first-void urine samples.  
Read all instructions prior to collection.

ONE PACKAGE CONTAINS

I. Colli-Pee® funnel  
A. Housing, B. Floater  
II. Collector tube  
C. Tube cap, D. Collector tube, E. Preservative 3 mL UAS™

CAUTION

Do NOT wash your intimate zone thoroughly before urinating.  
Do NOT urinate 1 hour prior to use.  
Do NOT use if the packaging is damaged.For single use only.  
Preservative should remain inside collector tube. Do not drink the preservative. Wash with water if preservative comes into contact with skin or eyes.\*

User instructions

1. Unscrew the tube cap (C).
2. Push the collector tube (D) on the Colli-Pee® funnel.
3. Collect urine: The device will automatically collect the correct volume. There is no need to interrupt the urine stream. Excess urine that no longer fits into the collector tube, will flow into the toilet via the other end of the housing.
4. Disconnect the collector tube from the Colli-Pee® funnel.
5. Close the tube tightly. Tube cap (C) should be screwed on tightly.
6. Dispose the Colli-Pee® funnel according to local regulations for plastic waste.
7. Wash your hands.

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